




RESCUE SWAG INSTRUCTIONS

TAKE A DEEP BREATH

STAY CALM AND TAKE ACTION!

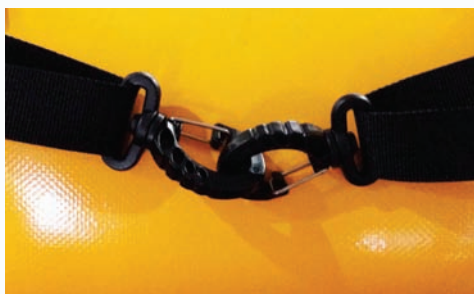

Attaching your Rescue Swag

The Original Rescue Swag is designed to attach to any item you should require, ensuring that your first aid supplies are with you when you really need them.

On the reverse of the kit you will find **attachment clips**, which provide durable, practical attachment points. Use the lockdown clips to fix your Rescue Swag in place, ensuring a snug fit to prevent movement.

You may also **extend webbing** straps to secure your kit, if clipping is not possible. Wrap the extended webbing around the object, clip and tighten. Ensure that webbing ends are securely tucked away and secured in this manner.



To use as a **SPLINT**:

1. Remove first aid packs from the Rescue Swag and zip it closed.
2. Lengthen the straps.
3. Wrap the Rescue Swag carefully around the limb as required - you will configure your splint to suit the injury. Take care to position around other injuries, as shown.
4. Tighten the straps until the joint is immobilised or compression is achieved. Take care not to over tighten.



To use as a **SLING**:

1. Remove first aid packs from the Rescue Swag and zip it closed.
2. Lengthen the straps.
3. Make a roll with the Rescue Swag and secure with external velcro tabs.
4. Carefully place the injured limb inside the roll.
5. Pass the extended strap around your neck and clip to the opposite buckle.
6. The second strap may be passed around the back and clipped in to the remaining buckle for greater stability.



To use as a **WATER BAG**:

1. Remove first aid packs from the Rescue Swag and zip up the sides only.
2. Line the Rescue Swag using the supplied plastic liner.
3. Fill with water as required.

Please seek medical advice as soon as possible after an accident. The Original Rescue Swag and contents are not intended to replace professional medical advice or treatment.

Contact us for more information:

e. sales@rescueswag.com.au

p. 1300 289 849

www.rescueswag.com.au



Download the

FREE RESCUE SWAG APP!



We trust that this product will provide you with peace of mind while enjoying your favourite outdoor pursuits. The Original Rescue Swag is covered by a 12 month workmanship warranty. Should you have any concerns regarding the workmanship of this product, please contact us and we will be happy to assist you. Please note that componentry may change as the Rescue Swag evolves - contact us for further information.

The Original Rescue Swag - Outdoor & Remote
Component List : Compliant Safework Australia First Aid in the Workplace Code of Practice

Description	Quantity	Purpose
Rescue Swag Pouch Yellow, including Water Liner (Module 1)	1	Carry all medical goods and provides splint, sling & immobilisation function.

Module 1 : Burns/Scalds, Sprains & Snake Bite	Description	Quantity	Purpose
	AEROFORM Con. H/Weight Band. 10cm x 1.5M	2	Compression for snake bite, support bandage for sprains
	AEROBAND Calico Tri. Band. 110cm x 110cm	1	Immobilisation
	AEROPLAST Instant Ice Pack 80g	1	Pain and swelling relief of spider bite, sprains and strains
	AEROFORM Conforming Band. 7.5cm x 1.5M	1	Retain dressings
	AEROFORM Conforming Band. 5cm x 1.5M	2	Retain dressings
	AEROBURN Burn Gel Sachet 3.5g	5	Burn relief
	AEROPAD Low Ad. Dressing 7.5cm x 10cm	2	Covering of burns
	AEROPAD Low Ad. Dressing 5cm x 5cm	3	Covering of burns
	AEROBURN PE Burn Sheet 60cm x 90cm	1	Covering of burns
	Instruction Card	2	

Module 2: Major Injuries	Description	Quantity	Purpose
	AERO First Aid Leaflet	1	Instructions for providing first aid
	AEROPLAST Amputated Parts Bags Ass. x 3	1	Body parts storage
	First Aid Notebook with Pen	1	Recording injuries and accident details
	AEROPAD Low Ad. Dressing 10cm x 10cm	1	Covering of large wounds
	AEROPAD Low Ad. Dressing 7.5cm x 10cm	1	Covering of large wounds
	AEROSWAB Gauze Swabs 3pcs 7.5cm x 7.5cm	2	Cleaning skin and wounds
	AEROBAND Calico Tri. Band. 110cm x 110cm	1	Immobilisation, control bleeding
	AEROWOUND Wound Dressing No.14	1	Control bleeding and protect wounds
	AEROWOUND Wound Dressing No.15	1	Control bleeding and protect wounds
	AEROWOUND Combine Dressing 10cm x 20cm	1	Control severe bleeding
	AEROFORM Conforming Band. 5cm x 1.5M	1	Retain dressings
	AEROFORM Conforming Band. 7.5cm x 1.5M	2	Retain dressings
	AEROFORM Conforming Band. 10cm x 1.5M	1	Retain dressings
	AEROSHIELD Face Shield Disp.	1	Resuscitation
	AEROPLAST Thermal Blanket 127cm x 180cm	1	Shock management
	Scissors Sharp/Blunt 13cm S/S	1	Removal of clothing and cutting of bandages and dressings
	AEROPINS Safety Pins Bag of 12	1	Securing dressings or sling
	AEROGLOVE Nitrile Glove Powder Free x 2	3	Personal protection
	Instruction Card	1	

Module 3: Minor Injuries	Description	Quantity	Purpose
	AEROPAD Low Ad. Dressing 5cm x 5cm	3	Covering of small wounds
	AEROSWAB Gauze Swabs 3pcs 7.5cm x 7.5cm	3	Cleaning skin and wounds
	AEROPAD Eye Pads 5.5cm x 7.7cm	5	Covering of eye wounds
	AEROPLAST Plastic Strip 72mm x 19mm x 25	1	Minor wound covering
	AEROPLAST Fabric Strip 72mm x 19mm x 25	1	Minor wound covering
	AEROPORE Micro. Paper Tape 2.5cm x 5M	1	Securing dressings
	AEROAID Antiseptic Tube 25g	1	Wound treatment, soothe bites and stings
	AEROWIPE Cleansing Wipes Env of 10	1	Cleaning and disinfecting wounds
	AEROWASH Eye Wash Liquid Ampoule 15ml	10	Wound and eye irrigation
	AEROWIPE Alcohol Swab 65x30mm	10	Cleaning areas surrounding wounds
	AEROWIPE Povidone Iodine Swabs 60 x 33mm	6	Cleaning and disinfecting wounds
	Forceps 13cm Stainless Steel	1	Foreign object removal, dressing handling
	AEROPROBE Splinter Probes 3.7cm x 1	10	Splinter removal
	AEROGLOVE Nitrile Glove Powder Free x 2	2	Personal protection
	Instruction Card	1	
	Red Aluminium Torch Whistle attached to outer pocket	1	